

- ⚠ **WARNING: Important! Read and understand these instructions before installing the casters. If the casters are not properly installed, personal injury and damage to the bed could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your GF Health Products, Inc. authorized distributor immediately. DO NOT use substitute parts. Use only Supernal replacement parts. Non-Supernal replacement parts could cause personal injury and cause damage to the casters or bed.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

Tools Required: M10 wrench & Phillips-head screwdriver.

HOW TO INSTALL BED CASTER(S)

Collect necessary tools and replacement caster(s).

1. Tip bed on side or upside down to install the caster wheels. Locate 4 (four) plastic glides on bed frame.
2. Remove 4 (four) plastic glides from bed frame.
3. Lay large flat washer down around hole left from the removal of the plastic glide.
4. Install the the new caster.
5. Repeat on remaining three casters.
6. Once all casters are installed place bed upright.



#1



#2



#3



#4

- ⚠ **CAUTION: It is important to always check your casters for proper performance and maintain them on a regular basis.**
- ⚠ **CAUTION: It is important to always check your casters and all components of the bed for proper function before every use.**