

- ⚠ **WARNING:** Important! Read and understand these instructions before installing the casters. If the casters are not properly installed, personal injury and damage to the bed could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your GF Health Products, Inc. authorized distributor immediately. DO NOT use substitute parts. Use only Supernal replacement parts. Non-Supernal replacement parts could cause personal injury and cause damage to the casters or bed.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

Tools Required: M10 wrench & Phillips-head screwdriver.

HOW TO INSTALL BED CASTER(S)

Collect necessary tools and replacement caster(s).

1. Tip bed on side or upside down to install the caster wheels.
Locate 4 (four) plastic glides on bed frame.
2. Remove 4 (four) plastic glides from bed frame.
3. Install the new caster.
4. Repeat on remaining three casters.
5. Once all casters are installed place bed upright.



#1



#2



#3

- ⚠ **CAUTION:** It is important to always check your casters for proper performance and maintain them on a regular basis.
- ⚠ **CAUTION:** It is important to always check your casters and all components of the bed for proper function before every use.